**REF. 2138 289 98 M**

<b>FR</b>	<b>NOTICE D'UTILISATION - HUILE SILICONE PURIFIEE</b>	<b>2</b>
<b>EN</b>	<b>INSTRUCTION FOR USE - PURIFIED SILICONE OIL</b>	<b>3</b>
<b>DE</b>	<b>GEBRAUCHSANWEISUNG - GEREINIGTES SILIKONÖL</b>	<b>4</b>
<b>IT</b>	<b>ISTRUZIONI PER L'USO - OLIO DI SILICONE PURIFICATO</b>	<b>5</b>
<b>ES</b>	<b>INSTRUCCIONES DE USO - ACEITE DE SILICONA PURIFICADO</b>	<b>6</b>
<b>PT</b>	<b>INSTRUÇÕES DE USO - ÓLEO DE SILICONE PURIFICADO</b>	<b>7</b>
<b>NL</b>	<b>GEBRUIKSAANWIJZING - GEZUIVERDE SILICONENOLIE</b>	<b>8</b>
<b>DA</b>	<b>BRUGSVEJLEDNING - RENSET SILIKONEOLIE</b>	<b>9</b>
<b>FI</b>	<b>KÄYTTÖOHJEET - PUHDISTETTU SILIKONIÖLJY</b>	<b>10</b>
<b>SE</b>	<b>BRUKSANVISNING - REN SILIKONOLJA</b>	<b>11</b>
<b>TR</b>	<b>KULLANIM - PÜRİFİYE SİLİKON YAĞI</b>	<b>12</b>
<b>BR</b>	<b>INSTRUÇÕES DE USO - ÓLEO DE SILICONE PURIFICADO</b>	<b>13</b>
<b>ZH</b>	<b>使用说明 - 硅油 商品名: FCI</b>	<b>14</b>
<b>EL</b>	<b>ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ - ΚΕΚΑΘΑΡΜΕΝΟ ΕΛΑΙΟ ΣΙΛΙΚΟΝΗΣ</b>	<b>18</b>
<b>PL</b>	<b>INSTRUKCJA UŻYTKOWANIA OCZYSZCZONY OLEJ SILIKONOWY</b>	<b>19</b>
<b>SL</b>	<b>NAVODILA ZA UPORABO – PREČIŠČENO SILIKONSKO OLJE</b>	<b>20</b>
<b>SK</b>	<b>NÁVOD NA POUŽITIE - PURIFIKOVANÝ SILIKÓNOVÝ OLEJ</b>	<b>21</b>

**Figures** ..... **22**

**Symbols** ..... **23**

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EN

Instructions for use  
Date of revision of the instructions for use : 10/2018

## PURIFIED SILICONE OIL

### Description and packaging:

PURIFIED SILICONE OIL is designed for the retinal endo-tamponade after vitrectomy. This oil is available in two viscosities :

Viscosity (25 °C)	1000 cSt	5000 cSt
Refractive index (20°C)	1.404	1.404

Appearance viscous, transparent, odorless liquid

Formula POLYDIMETHYLSILOXANE

PURIFIED SILICONE OIL is supplied sterile. It is dry heat sterilized in its primary packaging (vial or syringe). This is then packaged in a pouch for a steam sterilization, to make it easier to handle in aseptic conditions.

Our purification process guarantees a pure silicone oil, free of low molecular weight components and synthesis residues.

### Indications :

The PURIFIED SILICONE OIL is indicated in prolonged tamponade after surgical treatment for severe retinal detachment, especially :

- retinal detachment with proliferative vitreoretinopathy,
- retinal detachment that complicate a diabetic retinopathy
- retinal detachment with giant tears,
- traumatic retinal detachment,
- secondary retinal detachment to a viral retinitis

### Mode of action :

PURIFIED SILICONE OIL acts as a mechanical internal tamponade that reapplies the retina against the choroid and allows healing. There is no diffusion or absorption of the silicone out of the eye.

The implantation duration of the PURIFIED SILICONE OIL should be controlled and adapted to the retinal state of the patient.

### Contraindications :

The use of silicon oil for internal is advised against for patients who:

- Are hypersensitive to silicon
- Are wearers of intraocular lenses (highly hydrophobic), particularly in the case of posterior capsulotomy

### Adverse effects :

As in any surgical operation, surgery concerning the posterior segment using silicon oil has risks that are best evaluated by the surgeon. FCI cannot be held responsible for complications arising from the use of silicon oil.

As in any type of surgery, there are risks linked to the material or to developments of the initial pathology. Potential complications associated with the implantation of PURIFIED SILICONE OIL include but are not limited to the following:

- Intraoperative complications :
  - Ocular hypertony
  - Ocular hypotony
- Postoperative complications :
  - choroidal detachment
  - Closed angle glaucoma
  - corneal epithelium edema
  - keratitis
  - secondary retinal detachment
  - cataract
  - cystoid macular edema
  - late emulsification
  - infection, endophthalmis

Unexpected side effects and complications related to PURIFIED SILICONE OIL must be reported to FCI.

### Instructions for use :

Before use, the individual pack preserving the product's sterility should be checked to make sure it is intact.

#### Injection

##### Vials

- Transfer in aseptic conditions the content of the vial into a sterile syringe
- Connect the PURIFIED SILICONE OIL syringe to the infusion cannula
- Inject slowly.

##### Syringe

PURIFIED SILICONE OIL could be injected directly

A white polyoxymethylene stopper protects the syringe from accidental expulsion of the piston. Remove it before use by rotating it a quarter turn.

- Connect the PURIFIED SILICONE OIL syringe to the infusion cannula
- Inject slowly.

#### Removal

When removing the PURIFIED SILICONE OIL make sure that there are no silicone blebs in the vitreal cavity.

### Warnings :

Do not use the PURIFIED SILICONE OIL above 72.5 psi (5 bars). Injection of the PURIFIED SILICONE OIL above 72.5 psi (5 bars) may cause rupture of syringe and result in serious injury.

PURIFIED SILICONE OIL is a single-use liquid and must not be resterilized. The sterility and purity of the product are only guaranteed for a single use. It should be stored at room temperature, in a dry place and must not be used after the expiration date shown on the package.

We recommend removing the silicon oil once the desired effects have been observed. Silicon oil may reduce the absorption of light rays which could cause damage to the retina in the case of photocoagulation. Silicon oil's refractive index is rather higher than that of intraocular fluid and causes refractive error, notably when using standard calculations for the strength of intraocular lenses.

An implant card is available in the product's packaging. It is intended to be provided to the patient and allows traceability of the implanted product.

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